Invacare® Matrix® ELITE E2S/E2D Back

en	Backrest User Manual	3
de	Rückenlehne Gebrauchsanweisung	9
fr	Dossier Manuel d'utilisation	15
it	Schienale Manuale d'uso	21
es	Respaldo Manual del usuario	27
pt	Encosto Manual de utilização	33
nl	Rugleuning Gebruiksaanwijzing	39
da	Ryglæn Brugsanvisning	45
no	Ryggene Bruksanvisning	51
sv	Ryggstöd Bruksanvisning	57

DEALER: This manual MUST be given to the user of the product.

USER: Before using this product, read this manual and save for future reference.

matrix[®]
SEATING SERIES

INVACARE
Yes, you can.

TRD0586 Rev A

en This manual may not be reproduced or reprinted either partly or completely without previous written consent from Motion Concepts or its statutory representatives.
This user manual is compiled from the latest specifications and product information available at the time of publication. We reserve the right to make any necessary as they become necessary. Any changes made to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. If you have any questions or concerns regarding the information provided in this manual, please contact our Customer Service Department for assistance.

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1.1 OVERVIEW

SAFETY SYMBOLS:

Proper set-up and safe use of your Invacare Matrix Elite E2S/E2D Back depends on your own good judgement and common sense, as well as that of your provider, caregiver and/or health professional. The following symbols are used throughout this user manual to identify warnings, cautions and important notes. It is very important that you read and understand them completely.

 **CAUTION!** Failure to heed the cautions in this manual may result in damage to your Matrix Seating product.

 **WARNING!** Failure to heed the warnings in this manual may result in death, serious injury or damage.

 **IMPORTANT!** Important information to remember during installation, set-up and/or adjustment of you Matrix Seating product.

USER WARNING!

Improper use of this product may cause injury or damage. DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a health care professional or dealer before attempting to use this equipment.

IMPORTANT INFORMATION

The best way to avoid problems related to pressure injuries is to understand their causes and your role in a skin management program. Your therapist and physician should be consulted if you have questions regarding individual limitations and needs.

All back supports should be selected carefully. Working with your therapist and physician is the best way to assure that a cushion choice matches your individual needs. As the needs of the individual become more complex, the evaluation becomes more important.

 **IMPORTANT!** Please refer to the Wheelchair Owners/Operators Manual for additional safety information, warnings and cautions related to the operation of your wheelchair.

1.2 INTENDED USE

Our Invacare[®] Matrix[®] Elite E2S and E2D back is intended to provide back support for a wide variety of wheelchair users. It is designed to allow precise orientation within a wheelchair to ensure optimal postural support and positioning. The E2S/E2D Back support can be installed on a wide range of wheelchairs.

2.0 SAFETY INFORMATION

2.1 TRANSPORTATION WARNINGS!

⚠ WARNING! Risk of Death, Serious Injury or Damage

- Risk of severe injury when transporting the client within a wheelchair fitted with an Invacare® Matrx® Elite E2S/E2D back in a motor vehicle.

TRANSPORTING THE CLIENT IN A MOTOR VEHICLE WITHIN A WHEELCHAIR FITTED WITH A MATRX BACK

The following guidelines should be made available to all parties responsible for the transportation of the client, such as schools and transport providers.

- Wherever practicable Motion Concepts recommends that the client is transferred out of the wheelchair into a vehicle seat using a lap and diagonal seat belt or an appropriate child safety seat.

- The following back supports have been dynamically tested and meet the requirements of ISO 16840-4:2009, Wheelchair Seating- Part 4: Seating Systems for Use in Motor Vehicles; and Section 20, ANSI/RESNA WC-4:2017, Wheelchair Seating Systems for Use in Motor Vehicles:

- Invacare® Matrx® Elite E2S back with Removable Hardware and Elan Headrest
- Invacare® Matrx® Elite E2S back with Fixed Hardware and Elan Headrest

A forward facing frontal impact test was performed for each configuration using an ISO/RESNA Surrogate Wheelchair Frame (SWCF) and a Hybrid III Midsize-Adult Male Anthropomorphic Test Device (ATD). The SWCF was secured by a four-point, strap type wheelchair tie-down and the ATD was restrained by a surrogate three-point belt with a SWCF-Anchored Lap Belt.

- Invacare® Matrx® Elite E2S/E2D backs used to transport a client **must** be equipped with a Headrest Support (i.e. restricted to back heights 14in (36cm) and higher); and may **only** be used in conjunction with wheelchairs that comply with the performance requirements of RESNA WC19 or ISO 7176-19.

- For safe transportation of a client within a wheelchair in a motor vehicle, refer to the user manuals of the wheelchair and tie-down and occupant restraint system (WTORS)- see additional information below.

BEFORE ENTERING THE VEHICLE CHECK THE FOLLOWING:

The Matrx Back must be securely fastened into the wheelchair as described in the user manual.

Posture Belts & Harnesses should remain fitted during transportation, although they are designed to provide postural support only, they are NOT intended for safety during transportation.

A Head Support must be securely fitted and correctly adjusted close to the rear of the client's head during transportation, to reduce the risk of whiplash injuries.

WHEELCHAIR TIE-DOWN & OCCUPANT RESTRAINT SYSTEM (WTORS)

- **Wheelchair Tie-down:** The wheelchair must be securely tied down within the vehicle, in a forward facing position, in accordance with the wheelchair and tie-down manufacturers instructions.

- **Occupant Restraint:** A suitable occupant restraint system must also be fitted in accordance with the manufacturer's instructions. A minimum is a lap and diagonal

belt (lap belts on their own are not suitable). Restraints that loop over the shoulder and anchor to the vehicle floor should be avoided where possible, as they can cause a heavy downward load through the client during an impact, the preferred type anchor above and behind the shoulder as with a car seat belt. The lap section of the belt should fit snugly over the pelvis of the client and should not be able to ride up to the abdominal area. The upper torso section of belt should be in contact with the client's chest and fit over the shoulder(s), while not cutting into the neck or slipping off the shoulder(s).

i IMPORTANT! If the wheelchair and Invacare® Matrx® Elite E2S/E2D back is transported unoccupied, the wheelchair should be tied down with suitable equipment according to the manufacturer's instructions. If the equipment is dismantled for transportation, ensure that all parts are secured safely within the vehicle.

2.2 GENERAL SAFETY AND INSTALLATION WARNINGS!

⚠ WARNING! Risk of Death, Serious Injury, or Damage

Lighted cigarettes dropped onto an upholstered seating system can cause a fire resulting in death, serious injury or damage. Wheelchair occupants are at particular risk of death or serious injury from these fires and resulting fumes because they may not have the ability to move away from the wheelchair.

- DO NOT smoke while using this wheelchair

⚠ WARNING! Risk of Serious Injury

When establishing the mounting position for the Invacare® Matrx® back, ensure the stability of the wheelchair is not compromised. Adjusting the forward and aft position and/or angle of the backrest will alter the user's center of gravity within the wheelchair. It may be necessary to reposition the front casters, rear wheels, back angle, tilt-in-space, recline position and/or seat depth before use.

⚠ WARNING! Risk of Serious Injury or Damage

- The procedures in this manual should be performed by a qualified technician
- The hardware provided is high strength and tested for durability. **Do Not substitute hardware.** Use only the original hardware supplied with your Matrx back
- Attaching hardware that is loosely secured could cause loss of stability resulting in serious injury or damage.
- After ANY adjustments, repair or service and before use, make sure that all attaching hardware is tightened securely.
- DO NOT install the Invacare® Matrx® back assembly onto back canes with an outside diameter greater than 1" (25mm) or less than 3/4" (19mm). Otherwise, injury or damage may occur.
- For power seating applications, the Fixed Mounting Hardware must be used.

⚠ WARNING! Risk of Injury

Invacare® Matrx® products are specifically designed and manufactured for use in conjunction with Invacare® Matrx® accessories. Accessories designed by other manufacturers have not been tested by Motion Concepts and are not recommended.

⚠ WARNING Risk of Injury

Skin condition should be checked very frequently after the installation of any new seating system. Working with you therapist, physician and service provider is the best way to assure that a seating choice meets your individual needs.

3.0 MATRX ELITE E2S/E2D BACK INSTALLATION

3.1 MATRX E2S/E2D INSTALLATION & SET-UP

i IMPORTANT! The installation and set-up of the Matrix Elite E2S and E2D Back **MUST** be performed by a Qualified Technician and/or Healthcare Provider.

The Matrix Elite E2S/E2D Back has been designed to install onto back canes that are up to 1" (25mm) wider or 1" (25mm) narrower than the nominal back width. The type of mounting hardware required will vary depending on several factors, including the needs of the end user, the type of wheelchair being used, and the height of the Elite E2 back shell being installed.

4.0 CARE AND MAINTENANCE

4.1 INSPECTION AND MAINTENANCE

⚠ WARNING! Risk of Injury or Damage

DO NOT continue to use this product if problems are discovered. Corrective measures can be performed at, or arranged through your service provider.

Weekly: Inspect **ALL** fasteners weekly to ensure that mechanical connections and attaching hardware are tightened securely.

Monthly: Perform a visual inspection of parts including hardware, brackets, upholstery materials, foam, and plastics for deformation, corrosion, breakage, wear or compression.

4.2 CLEANING/LAUNDERING:

⚠ WARNING! Risk of Injury or Damage

To avoid infectious exposure, inspect cushion thoroughly and properly clean and disinfect any materials soiled with bodily fluids.

⚠ CAUTION! Risk of Damage

- DO NOT use cleaning agents containing phenols, alcohols, bleaches or other abrasive materials. Otherwise product damage may occur.
- 1% Chlorine Solution used on a regular basis can diminish the life of the cover if not rinsed and dried properly. Do not use granules.
- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Keep clear of open heat sources during cleaning.

General Care and Cleaning (Light Soilage):

1. The outer cover may be easily wiped down with a damp cloth using a 0.1% Chlorine Solution (1,000 ppm).
2. Rinse the outer cover with clean water using a single use nonabrasive cloth.
3. Dry the outer cover thoroughly before re-fitting to foam.

Laundering the Outer Cover:

PRIOR to laundering, the inner foam cushion must be removed from the cover).

1. Carefully remove the cover by releasing the velcro hook & loop fasteners along the top, bottom and inside of the back shell.
2. Machine wash cover in warm water (60C) on the Gentle Cycle.
3. Drip Dry **or** Tumble Dry on Low Heat (Dry cover thoroughly before re-fitting foam)

NOTE: Do Not Dry Clean, Do Not Iron, Do Not Bleach

Cleaning the Back Shell/Hardware:

1. The back shell and hardware/components can be wiped clean with soap and water as necessary.

4.3 REUSE/RECYCLING

WARNING! Risk of Injury

Always have an authorized dealer inspect the product for damage or wear PRIOR to transferring the product to a different user. If any damage is found, DO NOT use this product. Failure to heed these warnings may result in severe injury.

This product is suitable for re-use. The outer cover (fabric and foam), back shell and mounting hardware must be inspected by a qualified dealer or healthcare professional before re-use. If any damage is found, DO NOT RE-USE

Clean and disinfect all components (cover, foam, hardware and back shell) in accordance with local government regulations, prior to re-use.

For further assistance and/or information on decontamination and disinfection, please consult your local Infection Control Office.

4.4 DISPOSAL

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

5.0 WARRANTY INFORMATION

WARRANTY

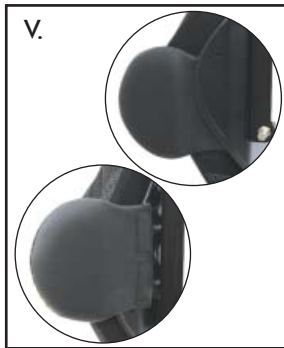
We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

6.0 ELITE E2S/E2D ACCESSORIES

6.1 ADDITIONAL E2 ACCESSORY OPTIONS (Sold Separately)

- I. PRIVACY FLAP (PFLAP)
- II. BACKBONE (PSIS/SacroLumbar Support)
- III. POSTURE FLEX (Abdominal Support) and QUAD RELEASE (QR) TAB
- IV. CHEST HARNESS INTERFACE
- V. EASYFIX LATERALS
- VI. SWING-AWAY LATERALS

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Advena Ltd.